



COVID-19 RAPID TESTING SUITE

Our Rapid, Accurate and Cost-Effective lateral flow tests (LFTs) for the detection of COVID-19 Antigen

Distributed by:

Quadrtech





CONTENTS

- 4-5. INTRODUCTION TO QUADRATECH DIAGNOSTICS
 - 4. *INTRODUCTION*
 - 5. *WHO ARE QUADRATECH DIAGNOSTICS?*

- 6-7. COVID-19 RAPID TESTING INTRODUCTION
 - 6. *OVERVIEW*
 - 7. *KEY ADVANTAGES OF COVID-19 RAPID TESTING*

- 8-15. COVID-19 RAPID ANTIGEN TESTS
 - 10. *WHY BUY ANTIGEN TESTS?*
 - 11. *HOW DO ANTIGEN TESTS WORK?*
 - 12. *HEALGEN ANTIGEN TEST KIT*

INTRODUCTION TO QUADRATECH DIAGNOSTICS

KNOWLEDGEABLE

We have 30 years of experience of supplying quality diagnostics products.

HELPFUL

We have a committed, friendly and trustworthy team to help our customers.

INNOVATIVE

We are dedicated to supplying state of the art diagnostics from industry leading manufacturers.

DEPENDABLE

We operate quality management systems to ensure the quality of products supplied.




WHO ARE QUADRATECH DIAGNOSTICS?

With 30 years trading experience, Quadratech Diagnostics Limited has always been committed to providing the highest quality state-of-the-art medical diagnostic and research products. Since 1990, we have expanded to encompass four main areas: Haemostasis, Bacterial Toxins, Rapid & POC Tests, and Parasitology.

We are a corporate member of the Institute of Biomedical Scientists (IBMS) and the British Society for Haemostasis and Thrombosis (BSHT). We are a recognised supplier to the UK National Health Service (NHS), the Irish Health Service Executive (HSE), private laboratories, the UK pharmaceutical industry and university research groups.

COVID-19 RAPID TESTING INTRODUCTION

Rapid testing is a powerful tool against COVID-19. As the pandemic continues, the key goal of any business is to ensure their business remains open while providing a safe workplace for their employees.

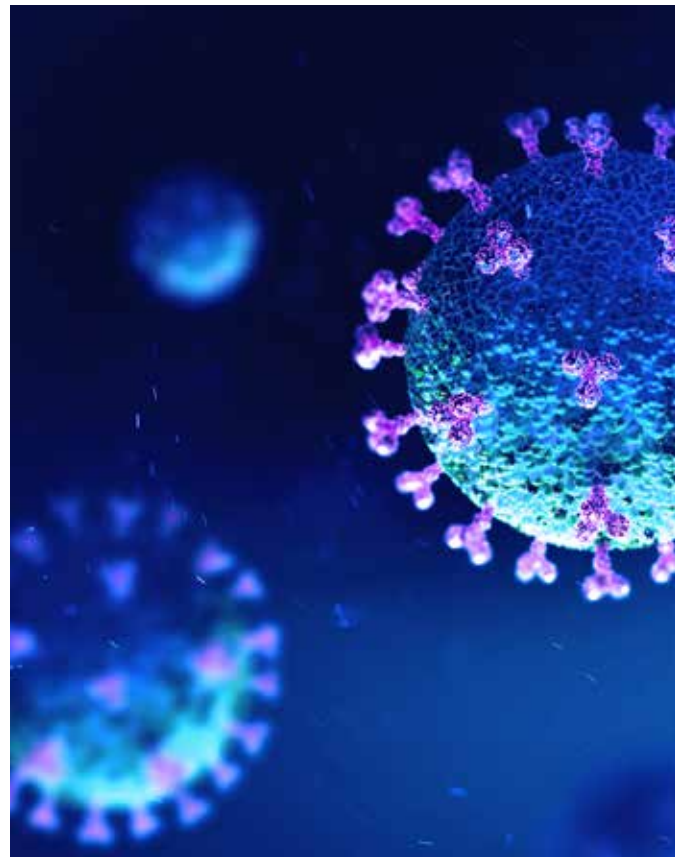

On-site testing enables businesses to proactively monitor the health of staff and identify potential outbreaks early to ensure you can remain open and operational. All whilst you look after the wellbeing of your most important asset - your staff.



KEY ADVANTAGES OF COVID-19 RAPID TESTING

Helps prevent outbreaks in the workplace ensuring continuity of service, manufacturing and trading.

Our tests determine if a person has the virus right now.

It's cost effective - eliminating the need for expensive lab-based testing.

Our Point-Of-Care tests deliver results on-site in 15 minutes.

Testing has become a key part of corporate social responsibility policy – it's hugely beneficial for the mental wellbeing of employers, staff and customers.

COVID-19 RAPID ANTIGEN TESTS

DETECT ACTIVE INFECTIONS


Often referred to as Lateral Flow Tests or Nasal Tests, rapid antigen testing represents a major advancement in bringing the pandemic under control.

Our market leading Healgen antigen tests look for pieces of proteins that make up the SARS-CoV-2 virus.

These tests help to determine if someone has an active COVID-19 infection.



COVID-19 RAPID ANTIGEN TESTS

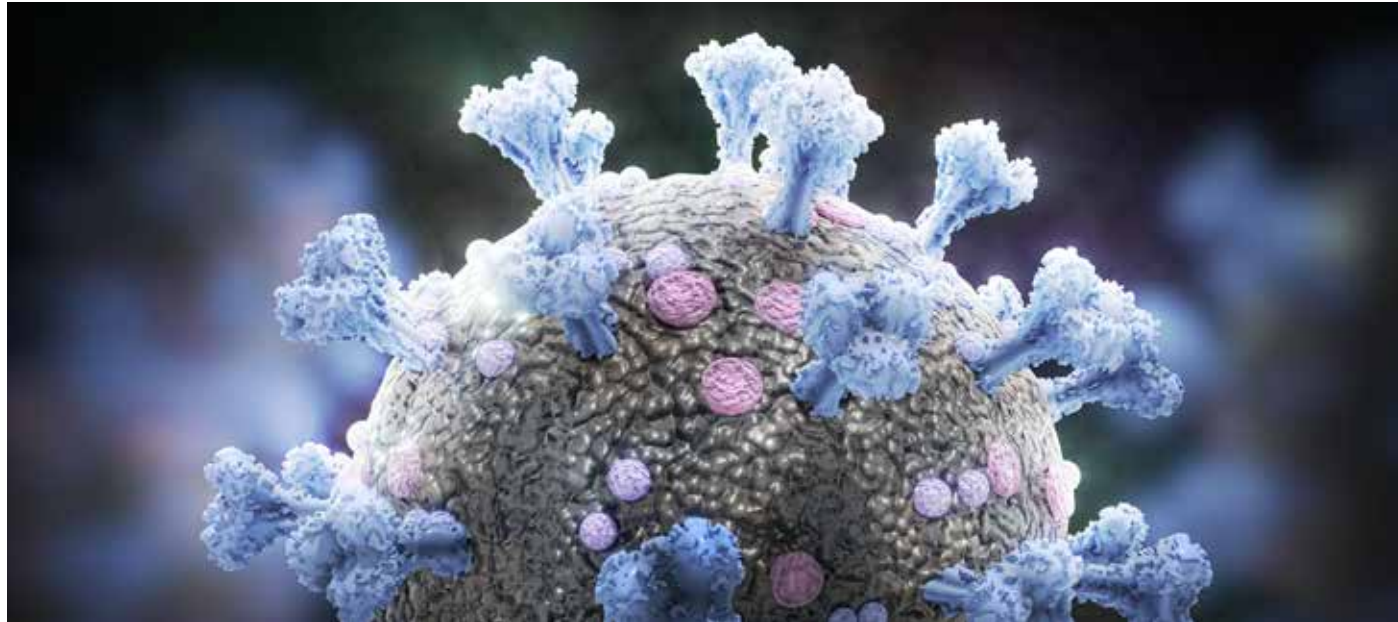

WHY BUY RAPID ANTIGEN TESTS?

Our rapid antigen tests will detect COVID-19 proteins in both asymptomatic and symptomatic patients.

Performing the test is simple and does not require expensive equipment to read results.

Test results are delivered on-site in 15 mins. No need to send samples away for analysis.

Unlike other antigen tests, our tests look for nucleocapsid protein making it the gold standard for rapid antigen testing.

The tests were evaluated in the USA and highly accurate with >99% Specificity, >96% Sensitivity & >98.5% Accuracy.

www.quadratech.co.uk


HOW DO RAPID ANTIGEN TESTS WORK?

1. A nasal swab is used to obtain a sample for testing.




2. Add extraction buffer to the extraction tube. Roll the swab 6 times inside the tube and leave to incubate for 1 minute.

3. Remove the swab and push the nozzle on the tube. This extracted solution is now the test sample to be used with the cartridge.

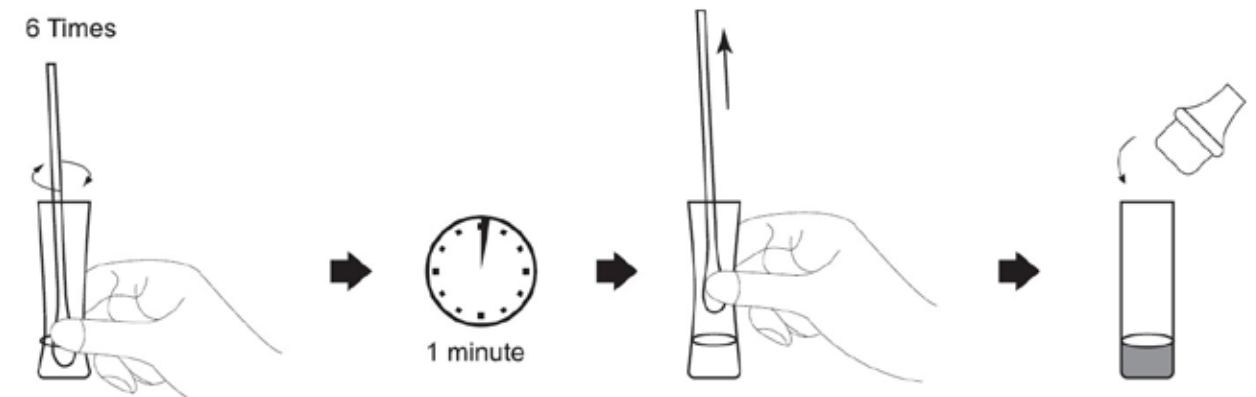

4. Add 4 drops of the test sample to the testing cartridge.

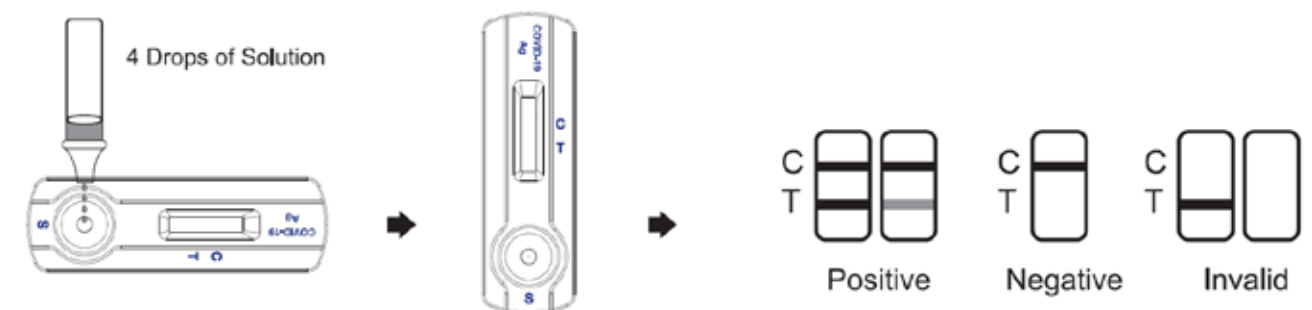

5. After 15 minutes, the test strip shows a positive or negative result.

www.quadratech.co.uk

HEALGEN RAPID COVID-19 ANTIGEN SELF TEST

QUADRATECH DIAGNOSTICS

Product Code: GCCOV-502a-H1

Kit Size: Single Tests

Brand: HEALGEN

CE Mark: For Self Testing



MHRA & UK GOVERNMENT VALIDATION

Evaluated and Validated by the UK's DHSC. Approved self-tests for use by schools, hospitals and the army. Used by the NHS in the UK.



EU APPROVED COVID-19 RAPID TEST

Formally approved by the EU's Health Security Committee for use across all EU member states.



DETECTS COVID-19 VARIANTS

Study by PHE and University of Oxford confirms variants, including Delta, are detectable.



PENDING EUA FROM FDA

Pending application for Emergency Use Authorisation with U.S Food & Drug Administration.



ISO13485 MANUFACTURING STANDARDS

HEALGEN manufacture in accordance to their ISO13485 Quality Management accreditation. They also hold many additional ISO accreditations.

Evaluated By:





**For more information
please contact:**

Quadraturetech Diagnostics Ltd
Station House, Cooksbridge
Lewes, East Sussex
BN8 4SW

T: 033 3321 2371

F: 033 3321 2372

E: info@quadraturetech.co.uk

www.quadraturetech.co.uk

Provider of Quality Medical Diagnostic & Research Products

Distributed by:

Quadraturetech

