


MICROSAFE® TUBE INSTRUCTIONS FOR USE 1/2

An air vent is strategically positioned on the sidewall of the tube to provide automatic venting and sample volume control.

Air vent regulates volume

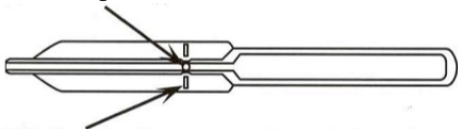

Fill line indicates total sample collected

WARNING: Filling is automatic. Never squeeze the tube while sampling.

MICROSAFE® TUBE INSTRUCTIONS FOR USE 2/2



1. Hold the tube horizontally and touch the tip of the MICROSAFE® Tube to the blood sample.

Capillary action will automatically draw the sample to the fill line and stop.

2. To expel the sample, align the tip of the tube with the sample target and squeeze the bulb.

If a sample won't expel, you probably didn't allow the tube to fill all the way. Touch the tip to the blood sample again and allow it to fill completely. Then align the tip with the sample target and squeeze the bulb.

